


BE THERE WITH WIDEX EVOKE AND SOUNDSSENSE LEARN



Tailor your hearing to YOUR needs – in the moment.

Different people have different preferences, even in the same situation. You can teach Widex EVOKE your preferences with a few taps on the SoundSense Learn feature in the Widex EVOKE app. SoundSense Learn uses real-time machine learning to learn and suggest a new, personalized setting.

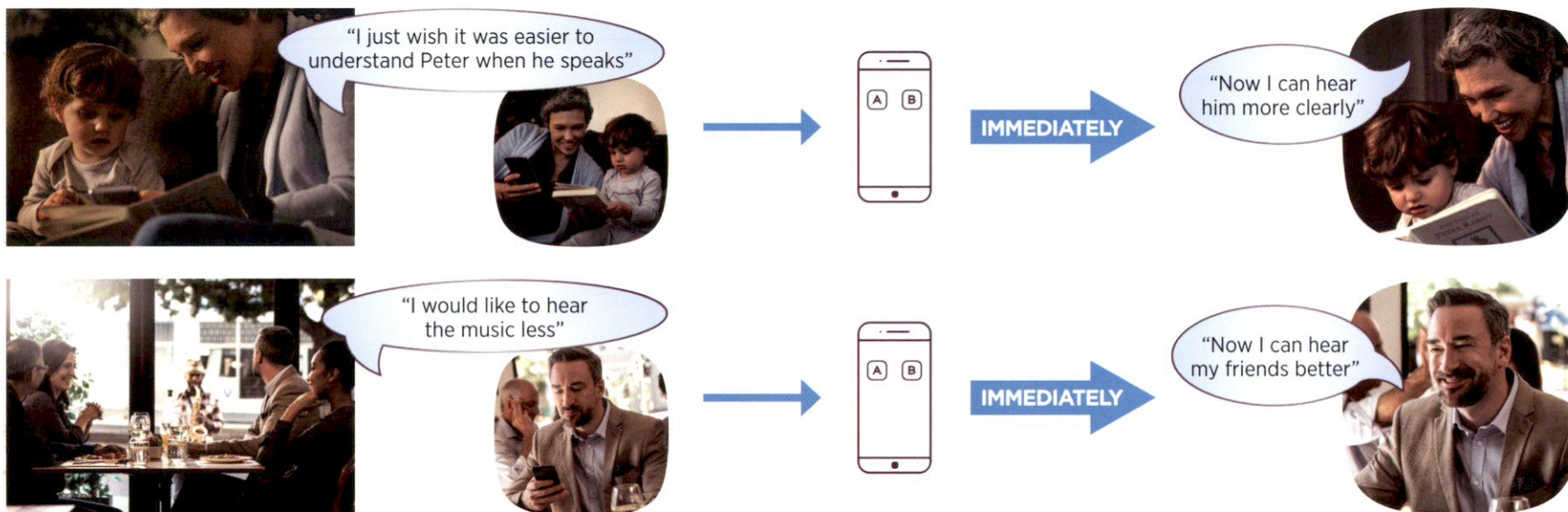

SoundSense Learn real-time machine learning offers an intelligent today and a smarter tomorrow.

INTELLIGENT TODAY

SMARTER TOMORROW

Hear your best in the moments that matter

Control what you want to hear and minimize the sounds you don't want to hear

Save time with fewer follow-up visits

Easy to use and personalized for you

Real-life insights for more informed follow-up visits

Automatic firmware updates with access to enhanced features

Continued learning through the Widex Cloud

Is my personal information protected? Yes, all information is sent anonymously to the Widex Cloud which is a secure and private server. Widex does not use any personal information in the process.

Widex Consumer Support: 1.844.497.8844 | widex.com

WIDEX EVOKE™
HEARING HAS EVOLVED



EVOKE SOUNDSense LEARN

IT'S EASY AS 1-2-3!


HEAR
YOUR BEST
AT ALL
TIMES WITH
EVOKE

In addition to superior sound quality and automatic performance, EVOKE lets you personalize your listening experience with the industry's only real-time machine learning tool: **SoundSense Learn**. Here's how you can take control to hear your best in any real-life hearing™ moment.

STEP 1 Getting started




Tap **Sound Menu** on the home screen of your EVOKE app and select **SoundSense Learn**.

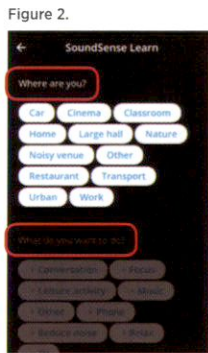


Select your environment and listening goals.

STEP 2 Improve your sound in real time

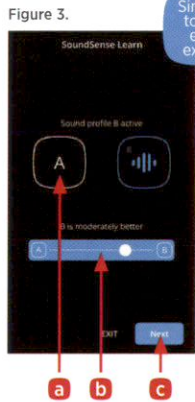


- a** Listen and compare the A and B sound profiles by tapping on each. Which do you like more?
- b** Move the slider to indicate your preference. The further you move it toward A or B, the more it means you like that sound.
- c** Select **Next** to listen to a new set of A and B sound comparisons and get closer to your preferred sound.

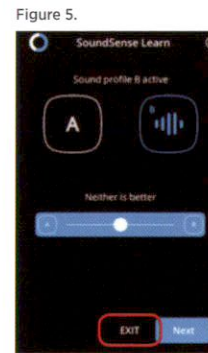

When you are happy with the sound, select **EXIT** at any time and your sound preference will be saved until you change programs or restart your hearing devices.

STEP 3 Save your settings as a personal program (Optional)

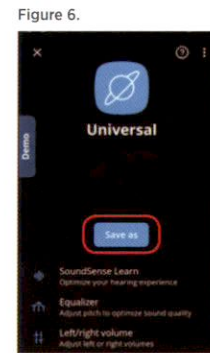

To use this sound setting again later, select **Save** to create a new personal program. It will be shown in your Programs list to access at any time in the future.



Name your program and select **Next**.



Select an icon to match your personal program and select **Create**.

Congratulations!
You've successfully personalized your hearing experience!